



Instructions for Zinnanti Surgical Reusable Tubing Kits and Adapters

CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- NON-STERILE: Product is used for non-sterile procedure or with sterile sleeve cover

CARE

All non-sterile tubing kit and adapter products:

14-4010D, 14-4010DHB, 14-4010HA, 14-4010HB, 14-4014, 14-4015, 14-4017, 14-4019, 13-4050, 13-4050DHB, 13-4050ESU, and adapters 14-4200 and 13-7000 are provided clean and packaged in clear plastic pouch with product label.

Please note: 13-4050, 13-4050DHB, 13-4050ESU are intended for single use only and **may be autoclaved one time**. Repeat autoclave cycles for these items will result in device failure.

14-4015, 14-4017 and 14-4019 are single use only due to collection of blood and fluid.

It is recommended that 14-4010D, 14-4010DHB, 14-4010HA, 14-4010HB, and 14-4010ESU are single use due to patient contact.

14-4014 may be reused if not coming into contact with patient. **It is recommended this tubing kits be replaced when there is visible wear, soil, or discoloration. Replacing the tubing kits after each clinic day when used multiple times is highly recommended to provide assurance that suctioned smoke and/or tissue does not come into contact with other patients, equipment or devices.**

Smoke evacuation adapters 14-4200 and 13-7000 are intended for multiple reuses and should be replaced if visibly damaged, discolored or soiled. The adapter may need to be replaced if suction is reduced by obstruction.

Carefully evaluate the device for wear or damage to packaging due to shipping or packaging failure. Please report any such damage or packaging failure to Zinnanti Surgical customer services (800) 459-1389.

For additional information, please contact us at:

Zinnanti Surgical Design Group Inc.

3540 Soquel Ave, Suite B

Santa Cruz, CA 95062

(800) 459-1389, FAX (800) 808-2728

info@zinnantisurgical.com