

Zinnanti Smoke-Evac Hyfrecator® Kit

14-4010HA, 14-4010DHB

Zinnanti Surgical Smoke-Evac Hyfrecator® electrode & tubing kit for electro fulguration/ablation of anal and lower genital tract lesions.

DIRECTIONS FOR USE / GENERAL OPERATION

Zinnanti Smoke Evac Hyfrecator electrode blades are provided sterile.
Zinnanti Smoke Evac Hyfrecator® tubing kit (14-4010HA), is provided non-sterile and does not require sterilization before use.

- 1) Place the soft plastic Y connector (14-4010HA) onto the distal end of the hand piece as shown (figure 1).
- 2) Place the electrode into the hand piece through the open end of the tubing kit connector end first (figure 2).
- 3) The electrode should be inserted all the way into the hand piece until the stem is seated tightly into the handpiece (figure 3).
- 4) The opposite free end of the tubing with blue colored connection boot can then be placed onto the smoke evacuator or tubing extension (figure 4).

INDICATIONS FOR USE

Consult Hyfrecator® instructions for use. Blades and tubing kit are designed for collection of smoke during electrofulguration/ablation of internal and external anal and genital lesions.



3540 Soquel Ave, Suite B, Santa Cruz, CA 95062
800-459-1389 . www.zinnantisurgical.com



EC REP

THD S.p.A.
Via Industria, 1
42015 Correggio (RE), Italy

FIGURE 1

Tubing set & Bovie hand-piece

PRODUCT# 14-4010-HA

FIGURE 2

Insert probe into handpiece

flute notch —

flute notch positioned upward may reduce flow

PRODUCT# 14-4010-HA

FIGURE 3

locate connection between probe and Y-connector tubing

PRODUCT# 14-4010-HA

FIGURE 4

Attach suction

PRODUCT# 14-4010-HA