

Tubing Adapter Kits Instruction for Use

Smoke-Evac Adapter with Sleeve

#14-4051 - fits Hyfrecator

#14-4051DHB - fits Bovie A941 and A942

#14-4051ESU - for use with extended electrodes
patent pending

FIGURE 1

Adapter kit with
6' gathered
accordion sleeve
in sterile chevron
pouch.

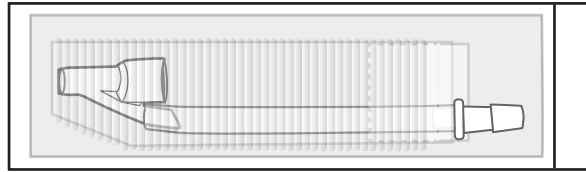


FIGURE 2

Peel open sterile
pouch.

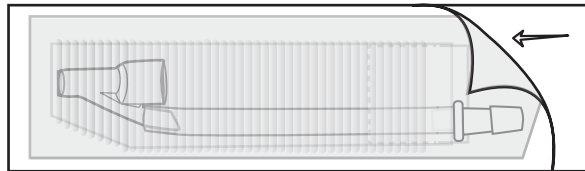


FIGURE 3

Insert handpiece
through card into Y
connector, at distal
end of pouch.

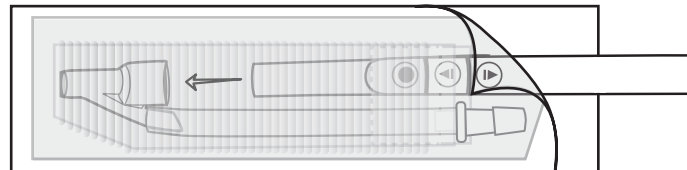


FIGURE 4

Connect suction
tubing.

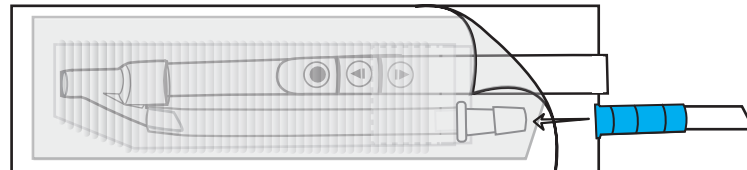


FIGURE 5

Remove sterile chevron
pouch. Pull card support
to extend 6' gathered
sleeve to cover full
length of cord & tubing.

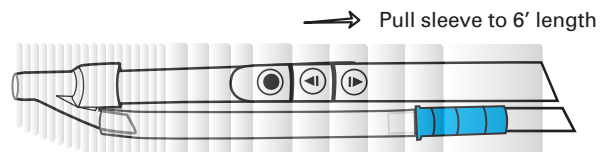


FIGURE 6

Insert special high-flow
electrode tip.

